

QuickChange™ Male Incontinence Wrap



Patient Profile

Non-ambulatory, incontinent male patients

Example use cases

- Retracted penis/enlarged scrotum
- Patients with skin breakdown
- Patients with an indwelling catheter
- “Open airing” with underpads



Infection prevention

Use in conjunction with an indwelling catheter to help reduce the risk of infection.



Catheter alternative

Use where external catheters are challenging.




Maintain skin integrity

Keeps urine off skin without using adhesives.



Minimize staff/patient contact

60-second check and change!

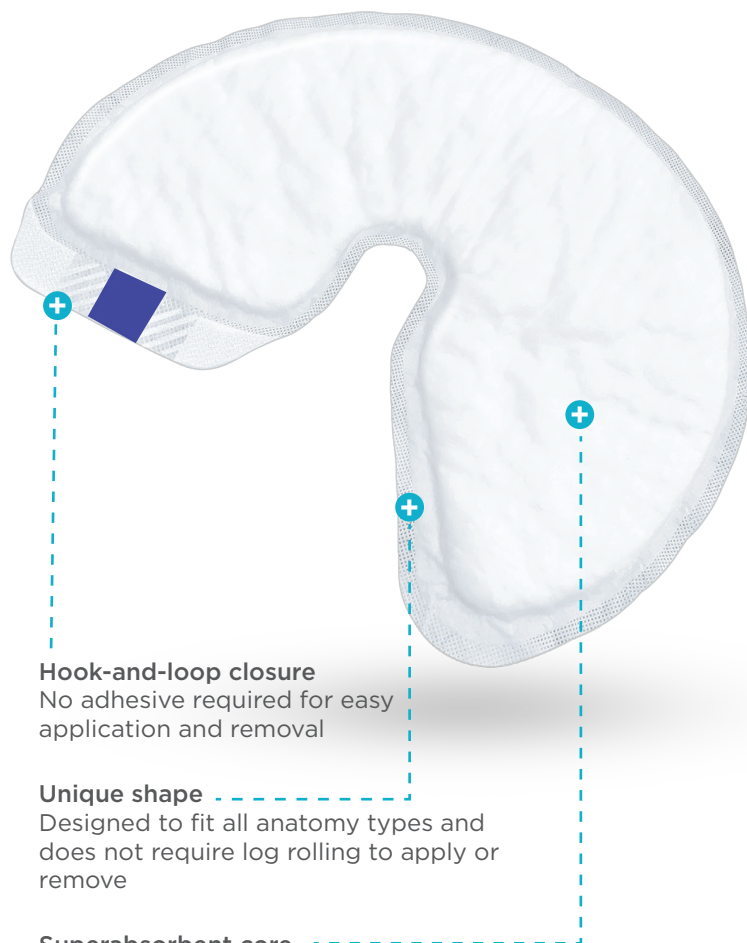

Hook-and-loop closure

No adhesive required for easy application and removal


Unique shape

Designed to fit all anatomy types and does not require log rolling to apply or remove


Superabsorbent core

Wicks moisture in and absorbs output before it pools onto skin or linens


How QuickChange is applied

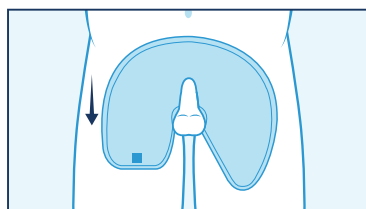

1. Place white side up, round edge towards the head & two flaps down toward the legs.

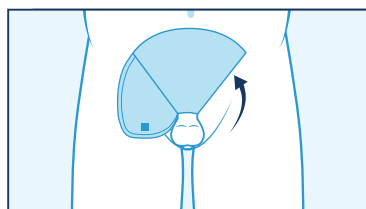

2. Fold pointed flap over penis at 45° and then fold straight flap over.

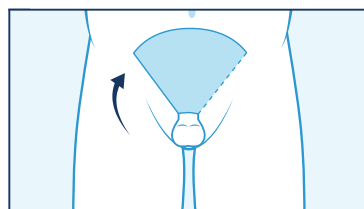

3. With two fingers, pinch hook fastener and inner flap together to ensure attachment.



Scan the QR code for additional applications and information


For additional information,
Contact your QuickChange Representative.
customerservice@quickchange.com

Additional applications and
instructions on backside of this page.


QuickChange™ Applications




Standard Penis (Use 1 Wrap)

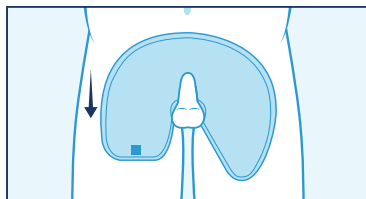

1. Place white side up, round edge towards the head & two flaps down toward the legs.

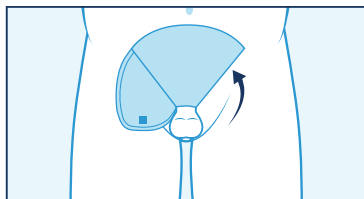


2. Fold pointed flap over penis at 45° and then fold straight flap over.

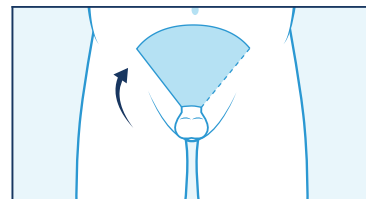


3. With two fingers, pinch hook fastener and inner flap together to ensure attachment.


Retracted Penis/Medium Scrotal Edema (Use 1 Wrap)

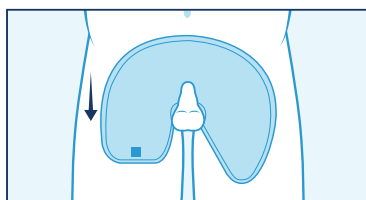

1. Place white side up, round edge towards the head & two flaps down toward the legs.

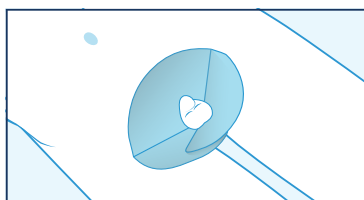


2. Slide pointed flap underneath scrotum, lift and scoop upwards and then fold straight flap over.

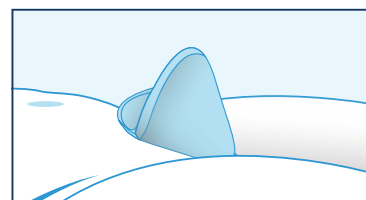


3. Pinch hook fastener and inner flap together to ensure attachment.

Large Scrotal Edema (Use 2 Wraps)

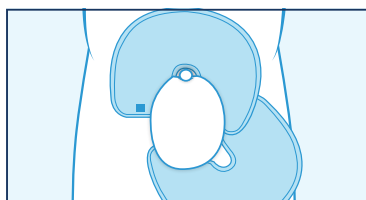


1. Position two wraps around scrotum & penis; secure a straight & a pointed flap together.

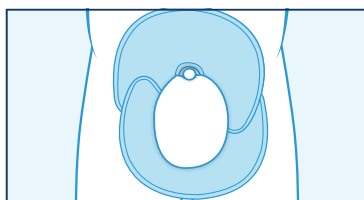


2. Pull other two flaps towards each other, creating a bowl shape underneath scrotum & pinch together to ensure attachment.

Bariatric (Use 2 Wraps)

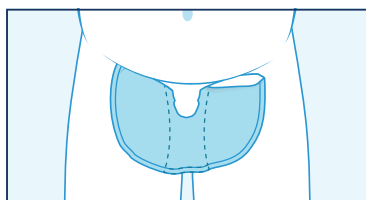


1. Place underneath scrotum, pull towards head to ensure contact, tuck between legs to catch runoff.

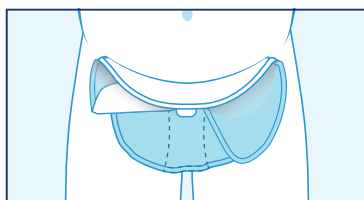


2. Place second wrap and fold in half downwards and lengthwise, make sure the center is adequately covered.

⚠️ Incorrect Usage

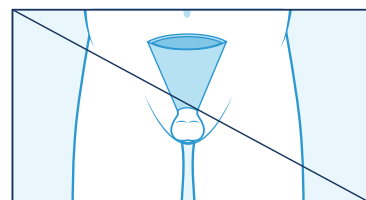

Do not roll into tight cone/funnel shape.

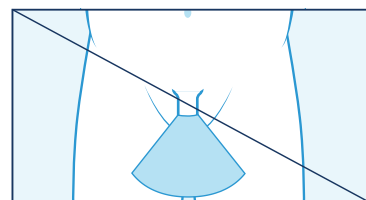

Do not leave the QuickChange pointed towards the legs.